

Methocel® 2%




This packing list contains important safety instructions and information on the use of the product. Read it carefully and keep it in a safe place for future use.

STERILE A

CE 0344

Distributed in Switzerland by
OmniVision AG, 8212 Neuhausen

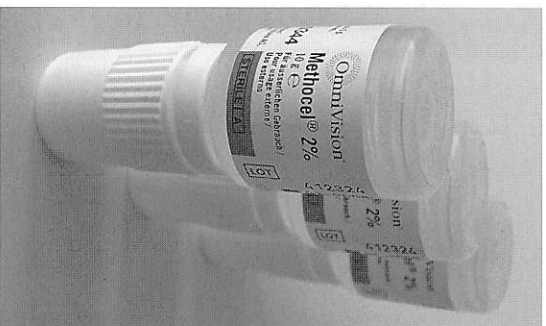

Solution for examination of the eye with diagnostic contact glasses. Manufactured under aseptic conditions (autoclaving and filling in germ-free zone).

1ml Methocel 2% contains 20 mg hydroxypropyl methylcellulose, 0.1 mg benzalkonium chloride (cons.), boric acid, amino hydroxy methyl propanediol, water for injecting.

Description

Methocel 2% is a highly viscose, colourless, transparent and isotonic solution of hydroxypropyl methylcellulose. The extreme purity of the hydroxypropyl methylcellulose allows the viscosity to be controlled precisely.

Indications

As contact liquid for fundus examination (gonioscopy and funduscopy) or in the treatment of the ocular fundus (e.g. argon laser trabeculoplasty) with the contact glass of the slit lamp.

Application

After thorough cleaning and disinfection, place 1-2 drops into the concave part of the contact glass.

When removing the contact glass, tip it to one side to prevent any suction. The epithelium should be checked for integrity after the examination.

Diagnostic contact glasses should only be used by a fully qualified and trained professional.

Limitations of use

Do not use if there is a likelihood of hypersensitivity to any of the active ingredients.

Possible side-effects

Temporary redness and swelling of the conjunctiva may occasionally result from the use of Methocel 2%.

Storage / Shelf life

Unopened packs of Methocel 2% are usable until the date printed on the package. Store at 15-25°C. Do not use any later than three months after first opening.



Note: Methocel 2% is easier to use if the bottle is stored upside down, resting on its wide cap.

Precautionary measures

Before opening for the first time, check to ensure that the warranty seal is intact. Keep out of reach of children.

For external use only – not for intraocular application.

Packaging

Dropper bottles, 3 x 10 g. Methocel is a registered trade mark of the Dow Chemical Company.

Manufacturer

OmniVision GmbH, D-82178 Puchheim
(Germany)

